




Sterile disposable lancets


To enable effective and comfortable capillary blood sampling

Virtually pain free

Electro-polished tri-beveled needle tip guarantees a smooth and sharp finish to obtain optimum blood samples with maximum comfort. Furthermore, the diameter of mylife™ Lancets amounts only 30G.

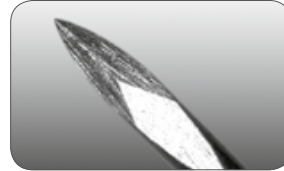


mylife™ Lancets
(150 × zoom)

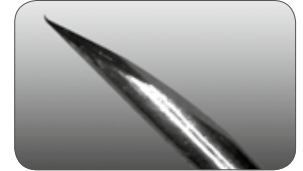


Competitors' Lancets
(150 × zoom)

Exposed Needle Length (ENL)


Needle is perfectly positioned to ensure consistent needle concentricity and exposed needle length.

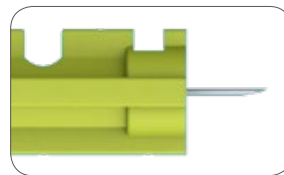


Handling steps for mylife™ Softlance

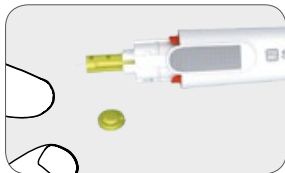

1. mylife™ Softlance lancing device and mylife™ Lancet after use.

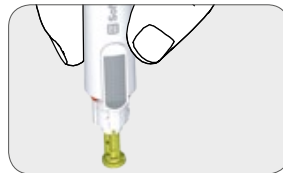

2. To prevent puncture wounds the lancet can easily be pierced into the cap.

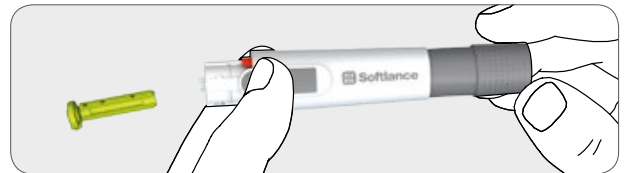


3. Hygienic and safe ejection of the lancet without touching it. Dispose the used lancet into an appropriate puncture-proof or biohazard container.




Handling steps for mylife™ AutoLance™



1. The mylife™ AutoLance™ with an automatic load and release function.

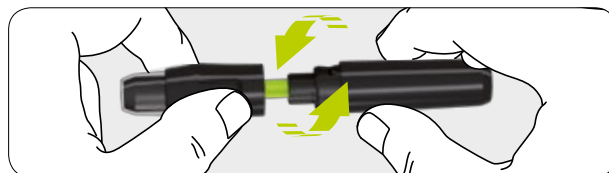

2. Practical bayonet lock and stop function for easy lancet replacement.

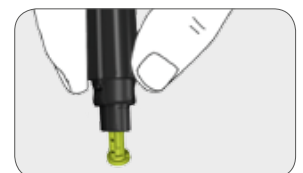

3. To prevent puncture wounds the lancet can easily be pierced into the cap. Safe disposal after use.



mylife™ Lancets high quality product range


Suitable for adults and children

mylife™ Lancets standard

High quality lancets for everyday use to ensure optimum blood flow. Designed to make blood sampling easier and more comfortable.

mylife™ Lancets multicolor

Same function as mylife™ Lancets standard, but available in seven different colors.

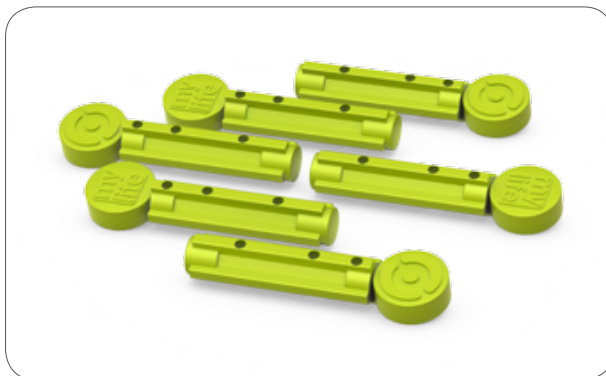

Maintains sterility for 5 years

Sterilizing dose determined in accordance with ISO 11137-2:2012

Ref. No.	Qty.	Description	Gauge	Needle Diameter	Point Length	Body Outer Diameter
7101030	200 pcs	mylife™ Lancets standard	30G	0.30 +/- 0.01mm	3.3 +/- 0.3mm	6.3 +/- 0.5mm
7101031	200 pcs	mylife™ Lancets multicolor	30G	0.30 +/- 0.01mm	3.3 +/- 0.3mm	6.3 +/- 0.5mm