



## **i** mylife™ SafetyLancets

Capillary blood sampling with reduced risk of accidental needle injuries

The innovative patented design makes capillary blood sampling safe and easy. The single release function and a sealed protection cap guarantee maximum security and protect patients as well as HCP's from cross-contamination or accidental finger lancing.




### Safety Features

- Safety seal ensures that the mylife™ SafetyLancet is unused
- Protective cap maintains sterility and prevents from contamination prior use
- The safely concealed needle before and after use prevents from needle stick injuries
- Inoperable after use thanks to single release function


### Comfort Features

- High speed penetration minimizes pain, making blood sampling feel like a soft touch
- Superior needle quality (tri-beveled needle tip) guarantees an optimum blood flow with maximum comfort
- No pre-loading or assembly prior use required


### How to use the mylife™ SafetyLancet

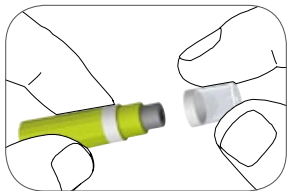


**1** Simply twist and remove the protective cap from the mylife™ SafetyLancet.

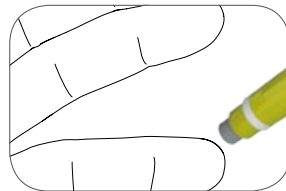

**2** Place the mylife™ SafetyLancet on the finger.

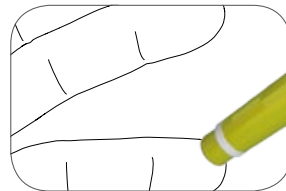

**3** Push the raised platform firmly against the chosen site to activate the lancet mechanism.

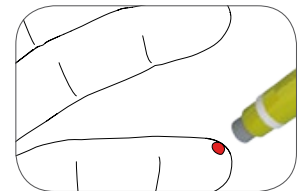

**4** The needle retracts immediately after use. Thanks to a single-use release function the mylife™ SafetyLancet cannot be re-used inadvertently.



**5** Dispose of the mylife™ SafetyLancet, in accordance with local regulations.





## mylife™ SafetyLancets

Sterile disposable safety lancets for a safe and comfortable routine medical blood sampling

Available in boxes of 200 pcs. with convenient slider lid



### Designed for blood glucose monitoring and other diagnostic tests in

- Diabetic clinics
- Hospitals
- Private homecare
- etc.

### Maintains sterility for 3 years

Sterilizing dose determined in accordance with ISO 11137

Sterilized by Co 60 Irradiation

Ref. No.	Qty.	Description	Gauge	Needle Diameter	Point Length	Body Outer Diameter
7100031	200 pcs	mylife™ SafetyLancets	28 G	0.36 mm	3.0 +/- 0.3 mm	10.5 +/- 2.0 mm

Customer Care: +91 99999 75157

More information on [www.mylife-diabetescare.co.in](http://www.mylife-diabetescare.co.in)

Ypsomed India Private Ltd // 10/61/1-F, Kirti Nagar Industrial Area // IN – New Delhi-110015

T +91 11 64607080 // F +91 11 25457080 // [info@ypsomed.co.in](mailto:info@ypsomed.co.in)