



mylife™ SafetyLancets

Capillary blood sampling with reduced risk of accidental needle stick injuries

The innovative patented design makes capillary blood sampling safe and easy. The single release function and a sealed protection cap offer security and protect patients as well as health care professionals from cross-contamination or injury. mylife™ SafetyLancets are available in two different versions: mylife™ SafetyLancets (28G/1.5 mm) and mylife™ SafetyLancets Comfort (30G/1.2 mm).

Safety features

- The safety seal visually indicates whether the mylife™ SafetyLancet has been used before or not
- The protective cap maintains sterility and prevents contamination prior to use
- Inoperable after use thanks to single release function



mylife™ SafetyLancets
(28G/1.5 mm)


Comfort features

- Superior needle quality (tri-beveled needle tip) supports an optimum blood flow with maximum comfort
- No pre-loading or assembly prior use required




mylife™ SafetyLancets
Comfort (30G/1.2 mm)


How to use the mylife™ SafetyLancet

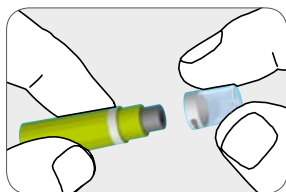


1 Simply twist and remove the protective cap from the mylife™ SafetyLancet.

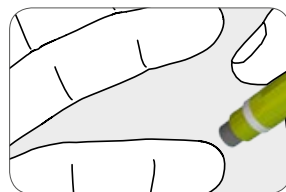

2 Place the mylife™ SafetyLancet on the finger.

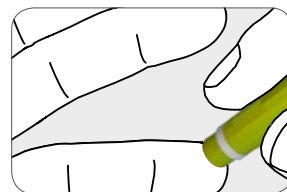

3 Push the raised platform firmly against the chosen site to activate the lancet mechanism.

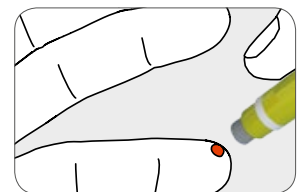

4 The needle retracts immediately after use. Thanks to a single-use release function the mylife™ SafetyLancet cannot be re-used inadvertently.





mylife™ SafetyLancets

Sterile disposable safety lancets for a safe and comfortable routine medical blood sampling

Available in boxes of 100 and 200 pieces with convenient slider lid



mylife™ SafetyLancets (28G/1.5 mm)



mylife™ SafetyLancets Comfort (30G/1.2 mm)

Designed for blood glucose monitoring and other diagnostic tests in

- Diabetes clinics
- Hospitals
- Homecare

Maintains sterility for 3 years

- Sterilising dose determined in accordance with ISO 11137
- Gamma-sterilised (Cobalt-60 irradiation)

Gauge	Needle diameter	Penetration depth	Description	Quantity	Ref. No.
28G	0.36mm	1.5 ± 0.2 mm	mylife™ SafetyLancets	100pcs	700009359
28G	0.36mm	1.5 ± 0.2 mm	mylife™ SafetyLancets	200pcs	7100031
30G	0.30mm	1.2 ± 0.2 mm	mylife™ SafetyLancets Comfort	100pcs	700009361
30G	0.30mm	1.2 ± 0.2 mm	mylife™ SafetyLancets Comfort	200pcs	700001495

India Ypsomed India Private Ltd. // 10/61/1-F, Kirti Nagar Industrial Area // New Delhi-110015 // info@ypsomed.co.in // www.mylife-diabetescare.co.in // Customer Care: 099999 75157

United Kingdom Ypsomed Ltd. // 1 Park Court // Riccall Road // Escrick // YO19 6ED // info@ypsomed.co.uk // www.mylife-diabetescare.co.uk // Customer Care: 0344 856 7820 (Calls to this number are free of charge from mobile phones and landlines, plus your phone company's access charge.)